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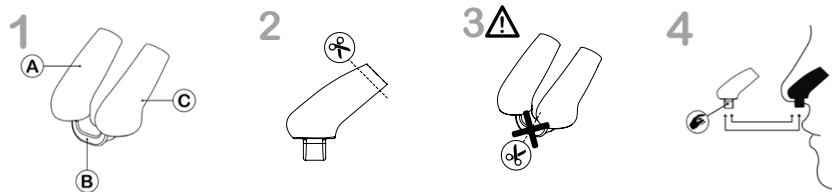
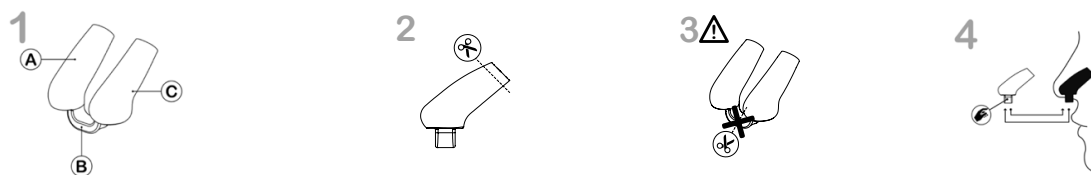
EN - Instructions for use

Keep this leaflet, you might need it later on.

1. When to use Pharmacleft Nasal Retainer®

Pharmacleft Nasal Retainer® is used in treatment of a deviated septum and/or nostrils. The retainer can be worn prior/after cleft lip surgery and nose surgery. Prior to surgery, the device helps to correct deformities of the nasal septum and nostrils to a more centred shape. After surgery, the *Pharmacleft Nasal Retainer®* is used to sustain the results of the surgical correction and further shape the nasal dome.

The *Pharmacleft Nasal Retainer®* shapes the nasal dome in a natural centred position and creates good airflow at the same time. The curved design follows the symmetrical shape of nostrils. Combined with the soft touch silicone the *Pharmacleft Nasal Retainer®* ensures optimal user comfort. The device has two tubes (fig.1; A&C) that are connected with a 'connection bridge' (fig.1; B) for safety.

2. Use of the Pharmacleft Nasal Retainer®

➤ Before first use

Make sure to fit the appropriate size in order to achieve the best results. If the retainer size is too wide or too small, please contact your treating surgeon for a replacement. To help surgeons decide which size number fits best for the patient, a *size set* can be ordered. Also, more variations of this product can be found on our web page www.pharmacleft.com/nasal-retainer.

If the tubes are too long (fig.1; A&C), the device might not be comfortable for the patient. Only if necessary, shorten the end of one/both tubes to fit the patients nose (fig.2). However, if the tubes are too short, the device loses some of its functionality. Cut only small pieces at a time. **DO NOT** cut the connection between both tubes! (fig.3) **DO NOT** damage the device while shortening! The *Pharmacleft Nasal Retainer®* **CANNOT** be used in case of damage and must be replaced.

If desired by local or hospital procedures, the *Pharmacleft Nasal Retainer®* can be sterilized in an autoclave before first use. Make sure the *Pharmacleft Nasal Retainer®* has cooled down to body temperature before insertion in the patient's nose.

➤ Insertion of the device

Before touching the device for insertion, please make sure to wash or disinfect your hands. Make sure the nostrils are clean and cleared before inserting the device. For comfortable insertion, potable water or a sterile saline solution can be used to decrease friction. When inserting the device, some small discomfort may be experienced. However, this discomfort should be limited and last no longer than a few minutes! Consult your treating physician if the discomfort persists.

To insert the retainer into the nose (fig.4), first insert one tube into the biggest nostril with the convex side (bulb) facing upwards. Then, gently insert the second tube into the smallest nostril.

➤ Use of the Pharmacleft Nasal Retainer

To prevent dehydration of the nasal mucosa and accumulation of crusts inside the retainer, drops of saline solution can be administered at least once a day. For use with young children/babies, the retainer can be attached with medical tape to the nose tip. Do not cover the breathing holes in the nostrils while doing so!

➤ How long to use

It is recommended to wear the device for at least 12 hours a day.

The *Pharmacleft Nasal Retainer®* should not be used for a period longer than 30 days. Please follow directives from your doctor at all times.

➤ Removal of the device

Remove the retainer from the nose (fig.4) by gently pulling the tube connection (fig.1; B) with your fingers (fig.4) until both tubes come out.

3. Maintenance

Use running potable water or a saline solution to clean the inside and outside of the retainer after every use and once a day minimum. A cotton swab can be used to remove crusts from the inside of the retainer. If necessary, put the device in boiling water, crusts will loosen immediately. Make sure the *Pharmacleft Nasal Retainer®* has cooled down to body temperature before reinsertion in the patient's nose.

4. Important warnings & precautions

- If the *Pharmacleft Nasal Retainer®* is used in young children under the age of 2: the retainer is best fixed to the columella skin using medical tape, especially when the patient cannot be looked after closely (for instance during sleep). **Do not cover the breathing holes** in the nostrils while doing so!
- Do not cut the connection between the tubes due to choking hazard. Do not disassemble.
- Store the device away from direct sunlight and at room temperature.
- Do not use beyond expiry date.
- Do not use in open wounds.

5. Side effects & contraindications

No side effects are known to us. Please contact your doctor if you experience any possible side effects.

No contraindications are known to us.

6. Sizes

To see the latest sizes and models, please visit www.pharmacleft.com/nasal-retainer

In case of a serious incident, please contact the manufacturer (www.pharmacleft.com) and the competent authority in your country.

Pharmacleft Nasal Retainer is a medical device.

Manufactured and distributed by Batojan (Pharmacleft) . Sportstraat 2 . 8551 Zwevegem . Belgium

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© Last update of IFU 06/2020 – v1.2